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Artikelnummer

Chargennummer

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Heated baby booties – Instructions for use

Dear Customer,

Thank you for buying this product. Below you will find important information about this product. Please read this carefully, in

particular the safety advice, as well as the advice on how to use and care for the product.

Please follow the instructions and retain the instruction sheet and the packaging!

LOT See packaging!

1. Scope of Application

These instructions for use are applicable for the following products:

REF 135-00 Heated baby booties Size 1, 0-6 months
136-00 Heated baby booties Size 2, 6-15 months

2. Contents

1 pair heated baby booties, Instructions for use

3. Purpose and intended Use

Heated baby booties can be used to warm up cold baby feet. They can be used when your baby is lying down or is in a baby carrier/sling and are not intended to be used for crawling and walking! Baby booties are available for babies from 0-6 months and 6-15 months.

4. Usage and Heating Instructions

Please remove the adhesive labels with heating instructions before using the booties and keep them with the packaging and Instructions for use! Please heat the heated baby booties in pairs.

The following options are available to heat them:

| Size 1, 0-6 months | Size 2, 6-15 months |
|---|--|
| <ul style="list-style-type: none"> • in the microwave at the lowest or medium defrost setting (max. 180 watt) for max. 1 minute (Follow the instructions of the microwave manufacturer!) | <ul style="list-style-type: none"> • in the microwave at the lowest or medium defrost setting (max. 180 watt) for max. 1.5 minutes (Follow the instructions of the microwave manufacturer!) |
| <ul style="list-style-type: none"> • in a non-preheated oven at a max. temperature of 110° C for max. 3 minutes | |
| <ul style="list-style-type: none"> • Place on a lukewarm radiator or tiled stove for a period of time | |

Immediately after heating the booties, shake and knead the booties to distribute the heat evenly! When heated for the first few times, you may find that there is slight moisture on the surface, but this will disappear after repeated use.

After heating, test the temperature of the inside of the booties to make sure that it is suitable for babies (max. 43° C) by putting it in the crook of your elbow! The heated baby booties can then be fixed onto the baby's ankle, using the Velcro straps.

5. **Caution! Important safety information!**

Before use, it is essential that you check the temperature of the child between the shoulder blades or at the nape of the neck. If it feels at all sweaty there, do not use the heated baby booties! In addition, do not use the booties when your baby is having their

midday nap or sleeping at night and do not use with children with reduced sensitivity to pain.

Please observe the specified directions on heating the bootees, as a longer heating time or higher temperatures may cause damage (risk of inflammation and burning)! Before heating the bootees in the microwave, check in the Instructions for use from the microwave manufacturer that heating grain products is permitted. Always supervise the heating process of the bootees and do not heat the bootees when moist! Let the bootees cool down before reheating them!

Ensure that the seams are always intact when using rape seed products for babies and toddlers! Loose rape seeds are a choking and suffocation hazard! If the seams have come open and loose seeds are leaking out, dispose of the bootees and the loose seeds immediately and make sure they are out of reach of children. Only use under direct supervision of an adult! If the product is used for babies under 12 months, the child's temperature should be checked regularly during the application to avoid overheating!

6. Care Advice and Storage

Do not wash the heated baby bootees in order to avoid the formation of rot/mould! Store the product in dry conditions! Protect from moisture!

7. Information about Shelf Life and Disposal

It is recommended to replace the heated baby bootees with a new pair after a maximum of 4 years, if used normally. The bootees can be disposed of as household waste (residual waste).

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