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Bio-Traubenkern-Kissen

- (DE)** Bio-Traubenkern-Kissen – Gebrauchsanweisung2
- (EN)** Organic Grape Seed Pillow – Instructions for Use6
- (FR)** Coussin de pépins de raisin bio –
Notice d'utilisation..... 10
- (IT)** Cuscino con vinaccioli bio – Istruzioni per l'uso 14
- (NL)** Bio-druivenpittkussen – Gebruiksaanwijzing..... 18
- (ES)** Cojín de pepitas de uva ecológico –
Instrucciones de uso.....22
- (RU)** Био-подушка с наполнителем из виноградных
косточек – Инструкция по применению27

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EN Organic Grape Seed Pillow – Instructions for Use

Dear Customer,

Thank you for buying this product. You have made a good choice by buying this natural product. Below we have compiled important product information for you. Please read this carefully, in particular the safety advice, as well as the advice on how to use and care for the product.

 Please follow the instructions and retain the instruction sheet and the packaging!

 **LOT** See Packaging!

1. Scope of Application

These instructions for use are applicable for the following product:

REF 675-00 Organic Grape Seed Pillow 12x12 cm

2. Contents

Organic Grape Seed Pillow, Instructions for Use

3. Intended Purpose and Use

The Organic Grape Seed Pillow can be used to apply momentary heat to increase feelings of wellbeing and especially for easing the body's aches and pains as a form of heat therapy. The product is only intended for private home use and can be used for babies, children and adults.

4. Areas of Application

The Organic Grape Seed Pillow is suitable as a warmer exclusively for the following areas of application:

- Stomach ache/Stomach cramps
- Constipation and flatulence
- Colds
- Generally warming cold areas of the body

5. Usage and Heating Instructions

Before heating the pillow, check for any unusual odors, mould stains or burn marks. Dispose of the product immediately if these are present. The following options are available to heat the pillow:

- Warm in a non-preheated oven at a max. temperature of 110 °C for max. 3 minutes
- Warm in the microwave at max. 180 watt for max. 1 minute (Follow the instructions of the microwave manufacturer!)
- Place on a lukewarm radiator or tiled stove for a longer period of time

Immediately after heating the pillow, distribute the heat evenly by shaking it! When heated for the first few times, you may find that there is slight moisture on the surface, but this will disappear after repeated use.

6. CAUTION! Important Safety Information

It is essential that you observe the specified directions on heating, as a longer heating time or higher temperatures may cau-

se damage (risk of inflammation and burning)! Before heating the pillow in the microwave, check that heating grain products is permitted in the manufacturer's instructions for use. After heating, test the temperature of the pillow to make sure that it is suitable for babies (max. 43 °C) by putting it in the crook of your elbow! The pillow should always be heated by an adult, only under supervision, and not heated when damp! Let the pillow cool down before reheating it! The product should not be used as a pillow or a seat under any circumstances.

If the symptoms persist or worsen, please seek advice from a doctor. Do not use the pillow if you suffer from an acute, chronic, inflammatory disease or an illness accompanied by a temperature, circulatory problems, fresh hematomas, bruises and open wounds or for individuals with a reduced sensation of pain. If there are existing skin conditions, heart disease, venous disease, diabetes or tumor disease, or if the user is pregnant please contact a doctor before use. Pay attention to intact seams! Loose grape seeds are a choking and suffocation hazard! Only use under direct supervision of an adult! If the seams have come open and loose seeds are leaking out, dispose of the product and the loose seeds immediately and make sure they are out of reach of children. If the product is used for babies under 12 months, the child's temperature should be checked regularly during the application to avoid overheating! All serious incidents occurring in connection with this product must be reported to the manufacturer and the responsible authorities in the user's country of residence.

7. Care Advice and Storage

Do not wash the Grape Seed Pillow in order to prevent the formation of rot/mould! Store the product in a dry place! Protect from moisture!

8. Information about Shelf Life and Disposal

With normal usage, the product should be replaced after 4 years. The product can be disposed of in household waste and recycled thermally without any problems. Please separate the individual packaging components and dispose of them in the relevant recycling system. Please be aware of regional disposal guidelines.

9. Material Composition

Fabric: 100% cotton (controlled organic cultivation)

Filling: Grape seeds (controlled organic cultivation)

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