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Natural Rubber Hot Water Bottle (optionally with Cover) - Instructions for Use

Dear Customer,

Thank you for buying this product. Below you will find important information about this product. Please read this carefully, in particular the safety advice, as well as the advice on how to use and care for the product.

Please retain these instructions and the packaging for future reference.

LOT See packaging

1. Scope of application

These instructions for use are applicable for the following products:

REF 644-00 Natural Rubber Hot Water Bottle with organic Cover, GOTS, small

645-00 Natural Rubber Hot Water Bottle with organic Cover, large

646-00 Natural Rubber Hot Water Bottle, small

647-00 Natural Rubber Hot Water Bottle, large

2. Contents

Natural Rubber Hot Water Bottle (cover included with product 644-00/645-00), Instructions for Use

3. Function and Intended Use

The Natural Rubber Hot Water Bottle is filled with warm water and provides the human body with warmth. The hot water bottle can be used for applying heat when feeling generally cold.

4. Usage Instructions

- Take the hot water bottle out of the cover, if applicable, before use and check the bottle and stopper for wear and damage prior to use. Next, put the hot water bottle back in a cover before filling.
- Never use boiling water when filling the hot water bottle (max.

50 °C).

- The formation of steam can cause an overfilled hot water bottle to swell and, in the worst case, burst. Make sure you only fill the bottle until it is two thirds full.
- Hold the hot water bottle by the neck in an upright position and fill slowly to avoid hot water splashing back. Do not fill using water from the domestic hot water system as this can considerably shorten the life of the hot water bottle. Instead we recommend that you heat the water in a pan or kettle, but do not allow it to boil. Wherever possible do not use very hard water or water with high copper levels.
- Expel air from the hot water bottle by lowering it carefully onto a flat surface until water appears at the opening. Screw the stopper sufficiently tight to ensure that there is no leakage from the hot water bottle. Fingertight should be adequate. Ensure that you do not overtighten the stopper. Finally, make sure the funnel is empty and the bottle is sealed by turning it upside down over a sink and squeezing it gently.

5. CAUTION! Important Safety Information!

Good to know!

When filling the hot water bottle, the water temperature must not exceed 50 °C. Natural rubber is very sensitive to high water temperatures and tends to become porous. For your own safety, and to ensure that you can enjoy your hot water bottle for as long as possible, please make sure you

**adhere to the maximum water temperature of 50 °C.
WARNING - HOT WATER BOTTLES CAN CAUSE
BURNS. AVOID LONG PERIODS OF DIRECT CON-
TACT WITH SKIN.**

If the skin is exposed to temperatures over 48° C for a long period of time, there is a risk of first degree burns occurring. Babies and small children have particularly sensitive skin. In this instance, the temperature should not exceed 43 °C.

- Particular caution is advised for use with babies/children. Use the crook of your arm to check whether the temperature of the covered hot water bottle filled with water is suitable for your babies/children (at a max. 43 °C)! Only use under direct supervision of an adult! If the product is used for babies under 1 year of age, the child's temperature should be checked regularly during the application to avoid overheating!
- The hot water bottle may not be used by people with limited sensitivity to heat and/or pain.
- This hot water bottle is made of natural rubber. Please be aware that some people may have an allergic reaction to natural rubber. Should you notice an allergic reaction while using the hot water bottle, stop using it at once.
- If there is a possibility that prolonged contact with the skin could occur, an adequate cover (included with product 644-00/645-00) should be used to prevent burns.
- Under no circumstance should the hot water bottle be used

as a cushion and be sat on. Do not use in combination with an electric blanket. Do not sleep with it, ideally only use it to warm the bed.

- Prevent contact with hot surfaces.
- Not suitable for microwave or oven use.
- Keep pointed or sharp objects away from the hot water bottle.
- Do not use any additives such as table salt or essential oils

6. Care Advice and Storage

Do not use abrasive cleansers, general purpose cleaners, or other cleaning agents! Simply wipe the hot water bottle with a damp cloth. When not in use, drain the hot water bottle completely and keep, with the stopper removed, in a cool, dry, dark place - preferably with the opening pointing downwards - and protect it from direct sunlight. Do not place anything on top of the bottle during storage. Prevent contact with oil or grease.

The cover included with product 644-00/645-00 can be removed and is machine-washable at 30 °C.

Recommendation: Wash the cover at a low spin. Place in a laundry bag for a particularly gentle wash.

Please be aware of the care symbols on the cover!

7. Information about Shelf Life and Disposal

It is recommended to replace the hot water bottle with a new one after a maximum of two years, if used normally. If there are any wear, tears, fragility or other damages present, do not use

the hot water bottle again in any circumstances.

The hot water bottle can be disposed of in household waste and be recycled thermally without a problem.

Please separate and sort the individual packaging components and put them in the correct recycling systems. Please be aware of regional disposal guidelines.

8. Material Composition

Hot water bottle with natural rubber

Cover (for product 644-00/645-00): 100 % cotton (from controlled organic cultivation)

9. Explanation of Symbols used

Manufacturer

Attention!

Article number

Batch number

Observe the instructions for use!

Keep out of direct sunlight

Our natural rubber hot water bottles are manufactured in accordance with the high quality standard BS 1970:2012

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