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Mikrowelle

Backofen

Organic Chest/Tummy Wrap – Instructions for Use

Dear Customer,

Thank you for buying this product. You have made a good choice by buying this natural product. Below we have compiled important product information for you. Please read this carefully, in particular the safety advice, as well as the advice on how to use and care for the product.

Please follow the instructions and retain the instruction sheet and the packaging!

LOT See Packaging!

1. Scope of Application

These instructions for use are applicable for the following products:

- REF** 672-00 Organic Chest/Tummy Wrap GOTS, Size 1 (1-12 months)
- 673-00 Organic Chest/Tummy Wrap GOTS, Size 2 (1-5 years)

2. Contents

Organic Chest/Tummy Wrap, Organic Grape Seed Pillow, Instructions for use

3. Intended Purpose and Use

The Organic Chest/Tummy Wrap can be to apply momentary heat to increase feelings of wellbeing and especially for easing the body's aches and pains as a form of heat therapy. This product is only designed for personal use at home and can be used for babies and children.

The flexible Velcro allows the wrap to be perfectly adjusted to your child's body and enables a secure and comfortable fit. The soft cotton filling inside caresses the skin and retains heat excellently. The grape seed pillow can be securely placed in the pocket provided. Grape seeds are able to retain heat for a very long period of time and slowly release it on to the skin.

4. Areas of Application

The Organic Chest/Tummy Wrap is suitable as a warmer exclusively for the following areas of application:

Chest wrap	Tummy wrap
Colds	Stomach ache/Stomach cramps Constipation and flatulence

5. Usage and Heating Instructions

Before heating the pillow, check for any unusual odors, mould stains or burn marks. Dispose of the product immediately if these are present. Please remove the Grape Seed Pillow from the wrap before heating.

The following options are available to heat the pillow:

- Warm in a non-preheated oven at a max. temperature of 110 °C for max. 3 minutes
- Warm in the microwave:
 - REF** Size 1, 672-00: max. 180 Watts for max. 1 minute
 - REF** Size 2, 673-00: max. 300 Watts for max. 1 minute
(Follow the instructions of the microwave manufacturer!)
- Place on a lukewarm radiator or tiled stove for a longer period of time.

Immediately after heating the pillow, distribute the heat evenly by shaking it! Subsequently insert the inner pillow into the wrap. When heated for the first few times, you may find that there is slight moisture on the surface, but this will disappear after repeated use.

6. CAUTION! Important Safety Information!

It is essential that you observe the specified directions on heating, as a longer heating time or higher temperatures may cause damage (risk of inflammation and burning)! Do not heat up the wrap! Before heating the pillow in the microwave, check that heating grain products is permitted in the manufacturer's instructions for use. The inner pillow should always be heated by an adult, only under supervision, and not heated when damp! Once the warmed inner pillow is back in its wrap, please test the temperature of the pillow to make sure that it is suitable for babies (max. 43 °C) by putting it in the crook of your elbow! Let the pillow cool down before reheating it! The product should not be used as a pillow or a seat under any circumstances.

If the symptoms persist or worsen, please seek advice from a doctor.

Do not use the pillow if you suffer from an acute, chronic, inflammatory disease or an illness accompanied by a temperature, circulatory problems, fresh hematomas, bruises and open wounds or for individuals with a reduced sensation of pain. If there are existing skin conditions, heart disease, venous disease, diabetes or tumor disease, please contact a doctor before use.

Pay attention to intact seams! Loose grape seeds are a choking and suffocation hazard!

Only use under direct supervision of an adult!

If the seams have come open and loose seeds are leaking out, dispose of the product and the loose seeds immediately and

make sure they are out of reach of children.

If the product is used for babies under 12 months, the child's temperature should be checked regularly during the application to avoid overheating!

All serious incidents occurring in connection with this product must be reported to the manufacturer and the responsible authorities in the user's country of residence.

7. Care Advice and Storage

The wrap (without the Grape Seed Pillow!) is machine-washable at 30 °C. Please observe the care symbols on the product! Do not wash the inner pillow filled with grape seeds in order to prevent the formation of rot/mould! Store the product in a dry place! Protect from moisture!

8. Information about Shelf Life and Disposal

With normal usage, the product should be replaced after 4 years. The product can be disposed of in household waste and recycled thermally without any problems. Please separate the individual packaging component and dispose of them in the relevant recycling system. Please be aware of regional disposal guidelines.

9. Material Composition

	Inner Pillow:	Wrap:
Material:	100% cotton (controlled organic cultivation)	100% cotton (controlled organic cultivation)

	Inner Pillow:	Wrap:
Filling:	Grape seeds (controlled organic cultivation)	---

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