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DE	Bio-Leinsamen-Kissen mit Bezug – Gebrauchsanweisung.....	2
EN	Organic Flax Seed Pillow with Cover – Instructions for Use.....	7
FR	Coussin biologique en graines de lin avec housse – Notice d'utilisation.....	11
IT	Cuscino bio di semi di lino con fodera – Istruzioni per l'uso.....	17
NL	Biologisch lijnzaad kussen met hoes – Gebruiksaanwijzing.....	22
ES	Cojín de semillas de lino ecológico con funda – Instrucciones de uso.....	26
RU	Натуральная подушка с наполнителем из семян льна с чехлом – Инструкция по применению.....	32

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Mikrowelle

Backofen

EN Organic Flax Seed Pillow with Cover

– Instructions for Use

Dear Customer,

Thank you for buying this product. You have made a good choice by buying this natural product. Below we have compiled important product information for you. Please read this carefully, in particular the safety advice, as well as the advice on how to use and care for the product.

Please follow the instructions and retain the instruction sheet and the packaging!

LOT See packaging!

1. Scope of Application

These instructions for use are applicable for the following products:

603-00 Organic Flax Seed Pillow for Babies with Cover,
13 x 13 cm

631-00 Organic Flax Seed Pillow, Rabbit, 18 x 18 cm

2. Contents

Organic Flax Seed Pillow with Cover, Instructions for Use

3. Intended Purpose and Use

The Organic Flax Seed Pillow can be used to apply momentary heat to increase feelings of wellbeing and especially for easing the body's aches and pains as a form of heat therapy. The pro-

duct is only intended for private home use and can be used for babies, children and adults.

4. Areas of Application

The organic Flax Seed Pillow is suitable as a warmer exclusively for the following areas of application:

- Stomach ache/Stomach cramps
- Constipation and flatulence
- Colds
- Generally warming cold areas of the body

5. Usage and Heating Instructions

Before heating the pillow, check for any unusual odors, mould stains or burn marks. Dispose of the product immediately if these are present. Please remove the inner pillow from the cover before heating.

The following options are available to heat the pillow:

REF	603-00 Organic Flax Seed Pillow for Babies with Cover, 13 x 13 cm	631-00 Organic Flax Seed Pillow, Rabbit, 18 x 18 cm
In a non-preheated oven	at a max. temperature of 110 °C for max. 3 minutes	at a max. temperature of 110 °C for max.5 minutes
In the microwave (Follow the instructions of the microwave manufacturer!)	at max. 180 watt for max. 1 minute	at max. 300 watt for max. 1 minute
Place on a lukewarm radiator or tiled stove for a longer period of time		

Immediately after heating the pillow, distribute the heat evenly by shaking it! Subsequently insert the inner pillow into the cover.

When heated for the first few times, you may find that there is slight moisture on the surface, but this will disappear after repeated use.

6. **CAUTION! Important Safety Information!**

It is essential that you observe the specified directions on heating, as a longer heating time or higher temperatures may cause damage (risk of inflammation and burning)! Do not heat up the cover! Before heating the pillow in the microwave, check that heating grain products is permitted in the manufacturer's instructions for use. The inner pillow should always be heated by an adult, only under supervision, and not heated when damp! Once the warmed inner pillow is back in its cover, please test the temperature of the pillow to make sure that it is suitable for babies (max. 43 °C) by putting it in the crook of your elbow! Let the pillow cool down before reheating it!

The product should not be used as a pillow or a seat under any circumstances.

If the symptoms persist or worsen, please seek advice from a doctor. Do not use the pillow if you suffer from an acute, chronic, inflammatory disease or an illness accompanied by a temperature, circulatory problems, fresh hematomas, bruises and open wounds or for individuals with a reduced sensation of pain. If there are existing skin conditions, heart disease,

venous disease, diabetes or tumor disease, or if the user is pregnant please contact a doctor before use. Pay attention to intact seams! Loose flax seeds are a choking and suffocation hazard! Only use under direct supervision of an adult! If the seams have come open and loose seeds are leaking out, dispose of the product and the loose seeds immediately and make sure they are out of reach of children. If the product is used for babies under 12 months, the child's temperature should be checked regularly during the application to avoid overheating! All serious incidents occurring in connection with this product must be reported to the manufacturer and the responsible authorities in the user's country of residence.

7. Care Advice and Storage

The cover can be removed and is machine-washable at 30/40 °C. Please wash the cover before the first use. Please observe the care symbols on the product! Do not wash the inner pillow filled with flax seeds in order to prevent the formation of rot/ mould! Store the product in a dry place! Protect from moisture!

8. Information about Shelf Life and Disposal

With normal usage, the product should be replaced after 4 years. The product can be disposed of in household waste and recycled thermally without any problems. Please separate the individual packaging components and dispose of them in the relevant recycling system. Please be aware of regional disposal guidelines.

9. Material Composition

	Inner Pillow:	Cover:
Material:	100% cotton (controlled organic cultivation)	100% cotton (controlled organic cultivation)
Filling:	Flax seeds (controlled organic cultivation)	REF 631-00: 100% cotton (controlled organic cultivation)

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